

RECLINER MECHANISM

Manual Synchro



Healthcare



POSITION UPRIGHT

Upright standard sitting position.



POSITION RELAXED - FIRST POSITION

Push-on-arm rest synchro action first position. Both back reclines and footrest rises simultaneously. The angle of the back and seat remains unchanged.



POSITION RELAXED - SECOND POSITION

Push-on-arm rest synchro action, further to second position. Back rest reclines further to provide a more relaxed position. Back and seat angle increases. Footrest position does not change from first position,

RECLINER MECHANISM

Manual with Pneumatic Gas Spring Assist*



Healthcare




POSITION UPRIGHT

Comfortable upright standard sitting position.




LEG REST POSITION

By pushing the lever on side of the chair forward, pneumatic gas spring is activated, raising the footrest. Back does not move, back angle remains the same.

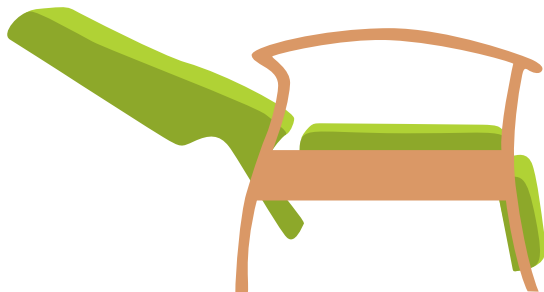

BACK REST POSITION

By pushing the lever on side of the chair backwards, pneumatic gas spring is activated, lowering the back rest. Footrest position is unchanged.

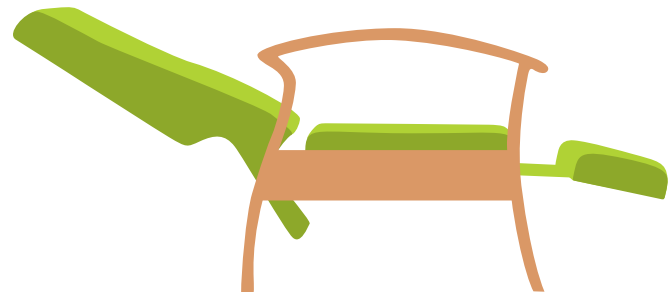

FULLY RECLINED POSITION

By pushing the lever on side of chair forward, pneumatic gas spring is activated, pushing down on the foot rest will cause the foot rest to lower. By pushing lever on side of chair backward, pneumatic gas spring is activated, release pressure on the back, and the back rest will begin to raise.

*Not available on all models.